

EpiTuub® Rapid Test System

*The most innovative completely closed one step
Rapid Test System*

Instructions for Patient

1. Collect a stool sample using the enclosed stool sampling paper:
 - a. Clean the bowl and flush the toilet two times. Unfold and lay the Sample Collection Paper directly on the top of the water in the toilet bowl (the paper should float above the water).



2. After bowel movement, proceed to Sample Collection Section.




*Easy sample collection
procedure without
unpleasant feeling.*


Sample Collection

1. Take the sampling tube and unscrew the sampling lid, keeping the sampling tube in a **vertical position** to prevent loss of solution.


Sampling Tube

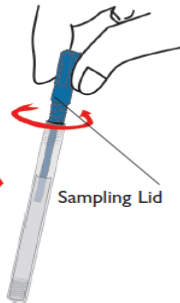

2. Hold the sampling lid by the Thumb Grip.
3. Use the tip of the sampling lid to collect a small amount of fecal sample at two or more sites. Only take the fecal sample that sticks to the sampling lid tip (never intentionally place any separate piece of fecal sample into the tube).

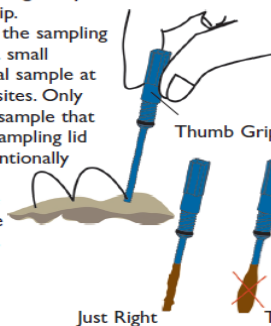


4. Insert and screw the sampling lid in a **vertical position**. Do not spill any solution from the tube.
5. Tightly seal the lid with the tube.
6. Flush the toilet.

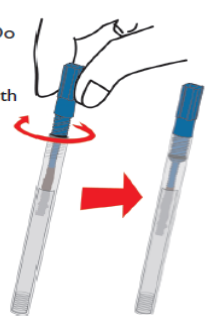


Test Procedure

1. Shake to dissolve the stool into solution.
2. Turn the sampling tube **upside down vertically**.
3. Remove the test strip from foil pouch.


Sampling Tube

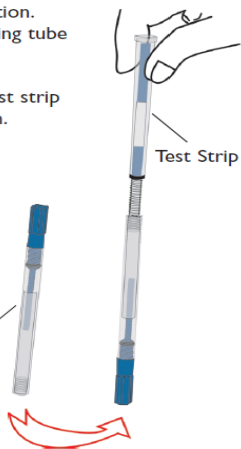

4. Insert and screw the test strip **in a vertical position** into the sampling tube by breaking the bottom seal of the sampling tube.
5. Allow the solution to flow into the bottom space of test strip, keeping the device **in a vertical position**.
6. You may soon see a red fluid moving across the white area of the test strip.

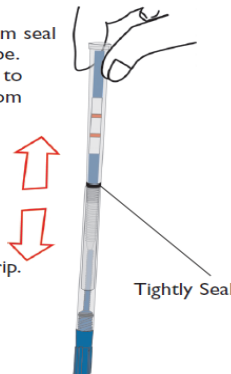

7. Read test results in five minutes. **DO NOT INTERPRET RESULT AFTER 10 MINUTES.**

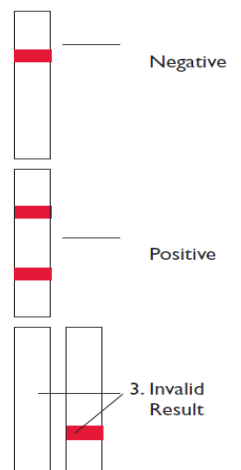

The EpiTuub® System is a fecal sample based rapid lateral flow test you can trust to diagnose a wide variety of medical conditions.

EpiTuub® Rapid Test Menu

Catalog No.	Product	Kit Size
KT-313	EPITUUB® Immunochemical Fecal Occult Blood Rapid Test Kit, CE IVD	30 Tests/kit
KT-916 KT-916.10	EPITUUB® Fecal Norovirus Antigen Rapid Test Kit, CE IVD	30 Tests/kit 10 Test/kit
KT-917	EPITUUB® Fecal Rotavirus Antigen Rapid Test Kit, CE IVD	30 Tests/kit
KT-918	EPITUUB® Fecal Adenovirus Antigen Rapid Test Kit, CE IVD	30 Tests/kit
KT-920 KT-921	EPITUUB® Fecal Calprotectin Rapid Test Kit, CE IVD	30 Tests/kit 10 Test/kit
KT-922 KT-922.10	EPITUUB® Calprotectin/iFOB Duo Rapid Test Kit CE IVD	30 Tests/kit 10 Test/kit
KT-926 KT-926.10	EPITUUB® Fecal Rota-Adeno Duo Antigen Rapid Test Kit, CE IVD	30 Tests/kit 10 Test/kit
KT-927 KT-927.10	EPITUUB® Fecal Giardia lamblia Antigen Rapid Test Kit CE IVD	30 Tests/kit 10 Test/kit
KT-928 KT-928.10	EPITUUB® Fecal Cryptosporidium parvum Antigen Rapid Test Kit CE IVD	30 Tests/kit 10 Test/kit
KT-929 KT-929.10	EPITUUB® Fecal H. pylori Antigen Rapid Test Kit, CE IVD	30 Tests/kit 10 Test/kit
KT-930 KT-930.10	EPITUUB® Fecal HAV Antigen Rapid Test Kit, CE IVD	30 Tests/kit 10 Test/kit
KT-932 KT-932.10	EPITUUB® Fecal Campylobacter Antigen Rapid test Kit, CE IVD	30 Tests/kit 10 Test/kit
KT-935 KT-935.10	EPITUUB® Fecal C. difficile GDH Rapid test Kit, CE IVD	30 Tests/kit 10 Test/kit

Supplied by:



Tel: +44(0)1235 431390
sales@oxfordbiosystems.com
www.oxfordbiosystems.com

Key Features of EpiTuub® Test System:

- ✓ Self-contained – Completely closed test system
- ✓ Simple assay Procedure – One Step
- ✓ Patented Rapid Test System
- ✓ Eliminates contamination to Lab & Specimen
- ✓ Advanced Rapid Test Methodology
- ✓ Clear test line interpretation