DiaMetra

SALIVA CONTROL

IVD

LOT
See external label

8°C
2°C 

$\Sigma = 3 \times 2\text{ml}$

REF DKO067

INTENDED USE

This product is intended for in vitro diagnostic use in the quality control of immunoassays using saliva samples

SAFETY PRECAUTIONS AND WARNINGS

For in vitro diagnostic use only. Do not pipette by mouth. Exercise all normal precautions required for handling laboratory reagents.

Since no method can offer complete assurance of the absence of any infectious agents, this material and all patient samples should be handled as though capable of transmitting infectious diseases and should be disposed of accordingly. Health and Safety Data Sheets are available on request.

STORAGE AND STABILITY

OPENED: Store refrigerated at 2 to 8°C. Product is stable until the expiration date printed on the external label. Dispense only the required amount of product necessary to perform the work. After use, discard any unused product. DO NOT RETURN ANY UNUSED MATERIAL TO THE ORIGINAL VIAL.

UNOPENED: Store refrigerated at 2 to 8°C. Material is stable until the expiration date printed on each individual vial.

MATERIALS PROVIDED

Saliva control:
DCE039 3 x 2 ml.(low, medium, high)

EQUIPMENT REQUIRED BUT NOT PROVIDED

Volumetric pipette

VALUE ASSIGNMENT

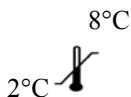
Each lot Batch of immunoassay controls is submitted to a number of reference laboratories and values are assigned from a consensus of results obtained by these laboratories. A mean value has been assigned to this lot of control using an ELISA method.

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