VDP

RID

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*For the in vitro determination of Vitamin D binding protein from
serum, plasma and urine*

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Table of content	Page
1. INTENDED USE	10
2. PRINCIPLE OF THE TEST	10
3. MATERIAL SUPPLIED	10
4. MATERIAL REQUIRED BUT NOT SUPPLIED	11
5. PREPARATION AND STORAGE OF REAGENTS	11
6. PRECAUTIONS	11
7. SPECIMEN COLLECTION AND PREPARATION	12
8. ASSAY PROCEDURE	12
PROCEDURAL NOTES	12
TEST PROCEDURE	13
9. RESULTS	13
10. LIMITATIONS	14
11. QUALITY CONTROL	14
EXPECTED VALUES	14
12. PERFORMANCE CHARACTERISTICS	14
LIMIT OF DETECTION	14
13. GENERAL NOTES ON THE TEST AND TEST PROCEDURE	15

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Lagerung/Storage: 2 - 8 °C

1. INTENDED USE

The *Immundiagnostik* Assay is intended for the quantitative determination of VDP in serum, plasma and urine. For in vitro diagnostic use only.

2. PRINCIPLE OF THE TEST

This test for the determination of vitamin D-binding protein (VDP) is based on a radial immunodiffusion in gel (RID, technique by Mancini). An antibody specific for the VDP is immobilised in gel on a radial immunodiffusion plate. This antibody forms a radial precipitate with the VDP from the sample. The precipitate is visualised as a ring and its diameter is directly proportional to the concentration of VDP in the sample.

Advantages of the RID-technique

- Only a small sample volume is required (10 µl)
- Non-radioactive procedure constitutes no hazard for personell

3. MATERIAL SUPPLIED

Catalogue No	Kit Components	Quantity
K 2313MTP	RID Platte, ready-to-use	17
K 2313ST	Standards, ready-to-use (8.7; 17.5; 26; 35 mg/dl)	4 x 100 µl

4. MATERIAL REQUIRED BUT NOT SUPPLIED

- 10 µl precision pipet
- Refrigerator (2-8°C)
- Deep freezer (-20°C)
- Humidity chamber

5. PREPARATION AND STORAGE OF REAGENTS

- The RID plate has been sealed in a protective pouch and is ready for use. At 2-8°C, it is stable until the expiry date printed on the kit box.
- The calibrators have been diluted to their working concentrations and are also ready for use. At 2-8°C, they are stable for four weeks after arrival. Immediate freezing to -20°C extends their stability until the expiry date printed on the kit box.

6. PRECAUTIONS

- For in vitro diagnostic use only.
- The substances could contain human source material which was tested and found to be non-reactive to HBsAg, anti-HIV-1/2, and anti-HCV. Since no method can offer complete assurance that hepatitis B virus, HIV-1/2, HVC or other infectious agents are absent, these reagents should be handled as if potentially infectious.
- Reagents should not be used beyond the expiration date shown on kit label.

7. SPECIMEN COLLECTION AND PREPARATION

Plasma or Serum

At 2-8°C, the samples are stable for a week, they should be deep frozen (-20°C), if they are stored for a longer period of time. All samples should be diluted 1:3 with 0.9% NaCl (i.e. 50 µl sample + 100 µl 0.9% NaCl).

Urine

The urine samples should be adjusted with 1N NaOH to a pH of 6-8 for storage. At 2-8°C, the samples are stable for a week, they should be deep frozen (-20°C), if they are stored for more than eight days. Samples containing more than 35 mg/dL VDP should be diluted 1:2 with 0.9% NaCl (i.e. 50 µl sample + 50 µl 0.9% NaCl).

8. ASSAY PROCEDURE

Procedural notes

- Do not interchange different lot numbers of any kit component within the same assay.
- The quality control guidelines should be observed.
- Incubation time, incubation temperature and pipetting volumes of the different components are defined by the producer. Any variations of the test procedure, that are not coordinated with the producer, may influence the test results. Immundiagnostik can therefore not be held reliable for any damage resulting from this.
- Carry out the assay with the actual manual delivered with the kit.

Test procedure

1. Pipette **10 µl** of calibrators, controls, and samples (plasma and serum diluted 1:3; urine undiluted or diluted 1:2 respectively; see unit Sample Preparation for more information) into the preformed holes in the RID plate.
2. Cover RID plate with cover and insert into the humidity chamber.
3. Incubate for **40-48 hours** at ambient temperature.
4. Evaluation of the assay by determination of the diameters of the precipitated rings.

9. RESULTS

The diameters of the precipitated rings are read from the millimetre scale that is imprinted on the RID plate or measured with a fine ruler. The evaluation is performed on semi-logarithmic paper. The measured diameters of the rings of the calibrators are linearly plotted on the x-axis against the VDP concentration on the logarithmic y-axis. The concentrations of the patient samples and of the controls can be read directly from this calibration curve.

Serum / Plasma

The results should be multiplied by the dilution factor of 3.

Urine

If the samples have been diluted, the result should be multiplied by the dilution factor of 2.

10. LIMITATIONS

Samples with VDP levels greater than the highest calibrator, should be diluted and re-assayed.

11. QUALITY CONTROL

Immundiagnostik recommends commercial control samples for internal quality control.

Control samples or serum pools should be analyzed with each run of calibrators and patient samples. Results, generated from the analysis of control samples, should be evaluated for acceptability using appropriate statistical methods. In assays in which one or more of the quality control sample values lie outside the acceptable limits, the results for the patient sample may not be valid.

Expected values

Plasma or serum:

20-55 mg/dl (M = 50.8 kD).

Deviations of the reference range may be present in the case of pregnancy (elevated) or acute disorders of the liver (decreased).

Urine:

below the detection limit.

Elevated levels may indicate a nephritic syndrome

12. PERFORMANCE CHARACTERISTICS

Limit of detection

The limit of detection of this test has been determined to be 5 mg/dl

13. GENERAL NOTES ON THE TEST AND TEST PROCEDURE

- This assay was produced and put on the market according to the IVD guidelines of 98/79/EC.
- The test components which are made of human serum are tested for Australia antigen and HIV and found to be negative. However, since no test method can offer complete assurance that infectious agents are absent, these reagents should be handled as recommended for any potentially infectious human serum or blood specimen. The normal precautions for laboratory working should be observed.
- Reagents of the test package contain sodium azide as a bactericide. Contact with skin or mucous membranes has to be avoided.
- All reagents in the test package are to be used for in-vitro diagnostics only.
- The reagents should not be used after the date of expiry (see label on the test package).
- Single components with different lot numbers should not be mixed or exchanged.
- The guidelines for medical laboratories should be observed.
- Incubation time, incubation temperature and pipetting volumes of the different components are defined by the producer. Any variations of the test procedure, that are not coordinated with the producer, may influence the results of the test. Immundiagnostik can therefore not be held reliable for any damage resulting from this.